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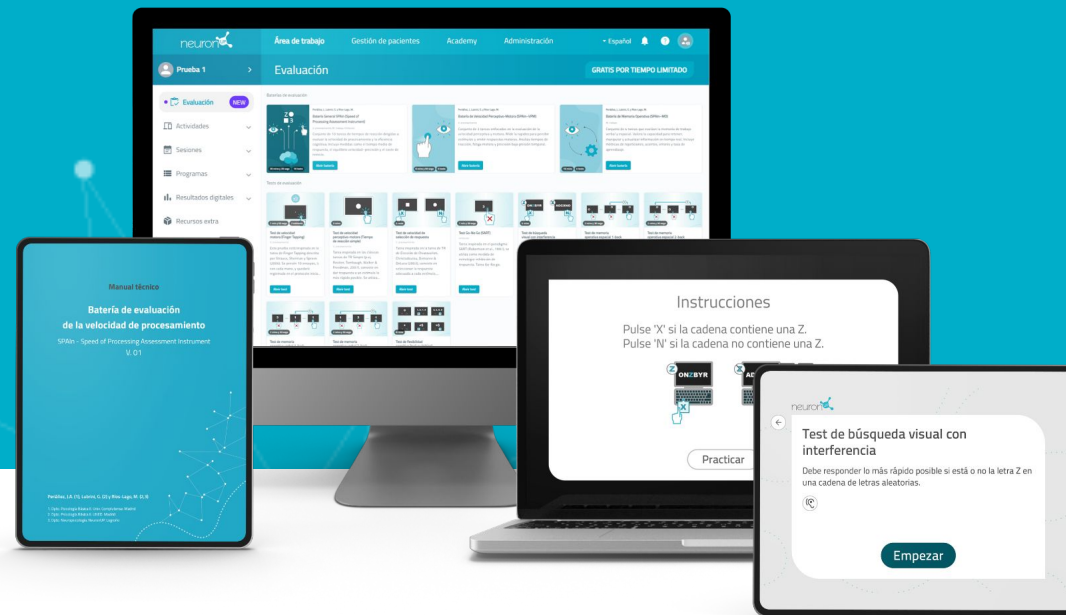
NeuronUP Assessment

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The screenshot displays the NeuronUP 'Arbeitsbereich' (Working Area) interface. The top navigation bar includes 'Working Area', 'Client Management', 'Academy', and 'Administration'. The user profile 'Manuel García' is visible in the top left. The main content area shows three test battery cards: 'SPAIIn (Speed of Processing Assessment Instrument)', 'Perceptual-Motor Speed Battery (SPAIIn-PMS)', and 'Working Memory Battery (SPAIIn-WM)'. The bottom section displays five individual assessment tests: 'Brief Test of Auditory Attention', 'Color Trails Test', 'Continuous Performance Test (CPT)', 'Stroop Test', and 'Stop Signal Task'. A chat input field is located at the bottom left.

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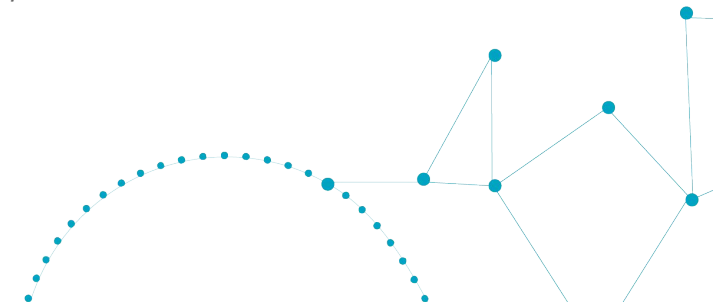
More details can be found in the pages on Creating and Configuring a Professional Profile.

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
The screenshot displays the NeuronUP interface. On the left, a navigation menu is visible with the 'Filter Patient' option highlighted by an orange box. The main area shows the 'Working Area' with a 'FILTER PATIENT' section. This section includes a search bar, a dropdown menu for 'NeuronUP2GO Access' set to 'All', and a list of patients: Patient 01, Patient 02, Chris Taylor, and Diane Thompson. A '+ New Patient' button is located at the bottom of the list. The status bar at the bottom right indicates 'No. of Patients: 14 / 14'. In the background, the 'Administration' view is partially visible, showing a search bar and a dropdown menu for 'Age' set to 'All'.

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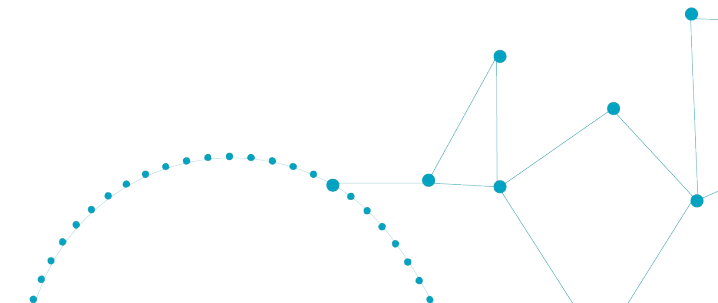The screenshot displays the NeuronUP Administration interface. The top navigation bar includes 'Working Area', 'Patient Management', 'Academy', and 'Administration'. The left sidebar contains 'Facility / Company', 'Payment Methods', 'Professionals', 'Invoice', and 'Plan management'. The main content area shows the 'Facility / Company' settings for 'NeuronUP', including sections for 'Relevant information' (Client View, Name, Type, Office, Specialty, Normal aging) and 'Billing Information' (VAT number, Phone Number, Address, Postal code). A 'Security and access' panel is highlighted, showing options for 'Restrict access by network' and 'Two-step verification'.

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Administrationsmodul

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The screenshot shows the 'Administration' tab in the NeuronUP interface. The main content area is titled 'Facility / Company' and contains a 'NeuronUP' logo placeholder and a 'Relevant information' section. The 'Relevant information' section includes fields for 'Client View: Name' (set to 'Patient') and 'How would you like to name your Clients?' (set to 'Patient'). Below these are fields for 'Type: Office' and 'Specialty: Normal aging'. A red box highlights the 'Edit relevant information' button, which opens a modal dialog box. The dialog box has a title 'Edit relevant information' and a close button. It contains five dropdown menus: 'Client View' (set to 'Name'), 'How would you like to name your Clients?' (set to 'Patient'), 'Type' (set to 'Office'), and 'Specialty' (set to 'Normal aging'). A red circle with the number '1' is next to the 'Name' dropdown, and a red circle with the number '2' is next to the 'Patient' dropdown. At the bottom of the dialog is a blue 'Save' button.

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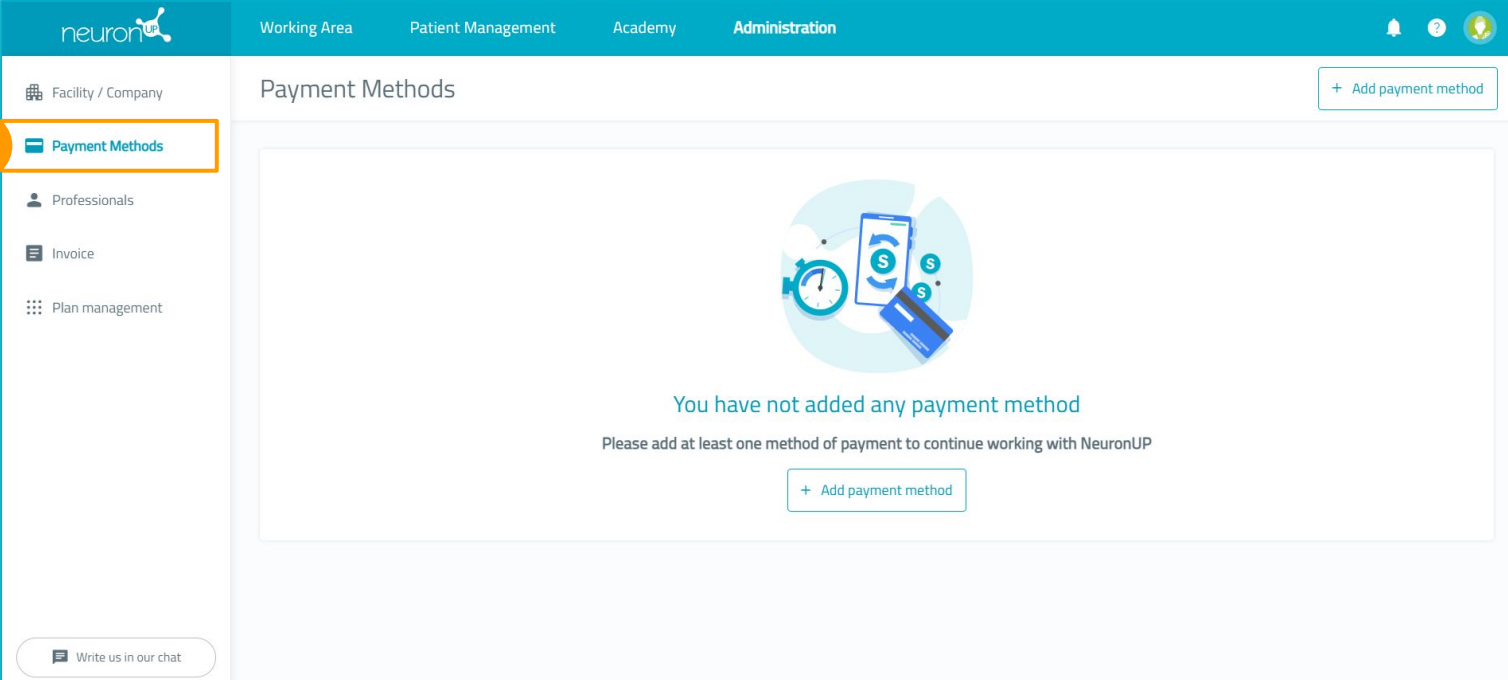
Administrationsmodul

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
The screenshot displays the NeuronUP Administration interface. The top navigation bar includes 'Working Area', 'Patient Management', 'Academy', and 'Administration'. The left sidebar contains a menu with 'Facility / Company', 'Payment Methods' (highlighted with an orange box and a '1' in a circle), 'Professionals', 'Invoice', and 'Plan management'. The main content area is titled 'Payment Methods' and features a central graphic of a smartphone, a clock, and a credit card. Below the graphic, the text reads: 'You have not added any payment method' and 'Please add at least one method of payment to continue working with NeuronUP'. Two '+ Add payment method' buttons are visible: one in the top right corner and one centered below the text. A chat box at the bottom left says 'Write us in our chat'.

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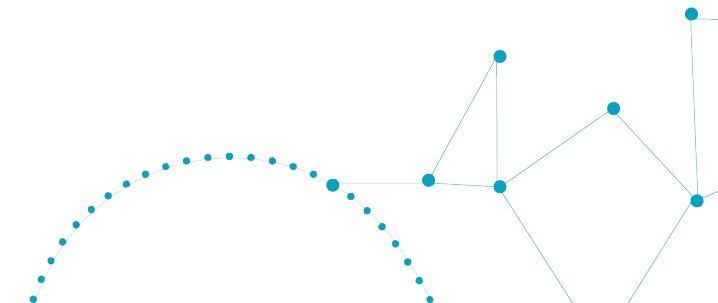Administrationsmodul

3. FACHKRÄFTE

The screenshot displays the 'Administration' section of the Neuron UP software. The left sidebar contains a menu with 'Professionals' highlighted, marked with a red circle and the number '1'. The main content area is titled 'Professionals' and shows a list of six professionals (Prof 01 to Prof 06) on the left. The right side of the main area displays a large card with a person icon and the text 'No Professional Selected', 'Select a professional from the list', and 'or create new'. A red circle and the number '2' highlight the 'No. of Professionals 61 / 70' and 'New Professional' button in the top right corner of the main area. The bottom of the main area shows 'No. of Professionals: 15 / 61'.

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Administrationsmodul

4. RECHNUNGEN


The screenshot displays the 'Administration' module in NeuronUP, specifically the 'Invoice' page. The interface is divided into a sidebar on the left and a main content area. The sidebar contains navigation options: Facility / Company, Payment Methods, Professionals, Invoice (highlighted with a callout '1'), and Plan management. The main content area features a search bar for 'Search Invoice' and two date filters: 'Start Date' (05/01/2025) and 'End Date' (11/30/2025). The main content area is divided into two tabs: 'Invoice' (highlighted with a callout '2') and 'NeuronUP2GO Months Assigned' (highlighted with a callout '3'). Below the tabs is a table with the following data:

Date	Professional	Patient	Months
Nov 13, 2025, 10:39:30 AM	anonymous	anonymous	1
Nov 7, 2025, 12:19:01 PM	anonymous	anonymous	1
Nov 6, 2025, 1:41:40 PM	anonymous	anonymous	1
Nov 5, 2025, 10:12:13 AM	anonymous	anonymous	1
Oct 28, 2025, 1:18:40 PM	anonymous	anonymous	1

At the bottom of the page, there is a chat button labeled 'Write us in our chat' and a status indicator 'No. of Assignments: 15 / 24'.

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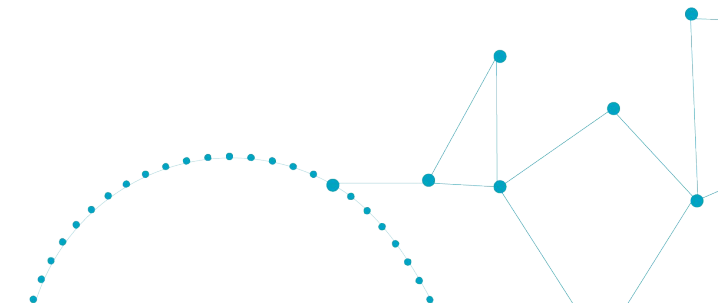Administrationsmodul

5. TARIFVERWALTUNG

The screenshot displays the NeuronUP Administration interface. The top navigation bar includes 'Working Area', 'Patient Management', 'Academy', and 'Administration'. The left sidebar contains menu items: 'Facility / Company', 'Payment Methods', 'Professionals', 'Invoice', and 'Plan management', which is highlighted with an orange box and a '1' in a circle. The main content area is titled 'Plan management' and shows 'Current Subscription' and 'My Plans'. Under 'My Plans', there is a 'Premium License (Free)' section containing 'Adults + Kids digital', 'Adults + Kids paper', and '10 professionals'. Below this, there are '60 extra professionals (Free)' and 'Test NeuronUP (Extra) (Free)'. A help box on the right asks 'How can we help you?' and provides contact information: '+34 601 71 96 99', '+1 (305) 424-8333', 'neuronup.us@neuronup.com', and a 'Write us in our chat' link. A chat button is also visible in the bottom left corner.

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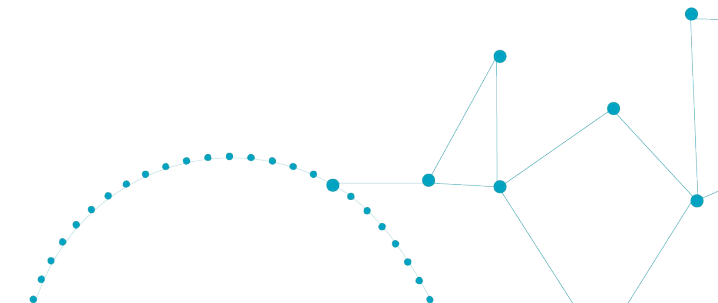
PROFESSIONAL-PROFIL ERSTELLEN

The screenshot displays the 'Administration' module of the Neuron UP system. The left sidebar contains navigation options: Facility / Company, Payment Methods, Professionals (highlighted with a blue box and a '1' in an orange circle), Invoice, and Plan management. The main content area is titled 'Professionals' and shows a list of existing professionals (Prof 01 to Prof 06) and a 'New Professional' button (highlighted with a blue box and a '2' in an orange circle). A modal window titled 'New Professional' is open, showing a three-step process: 1. Basic information (Name*, Surname*), 2. Access data, and 3. Relevant information (Optional). The 'Basic information' step is currently active. The form includes fields for Name and Surname, a photo upload section with a 'Upload Photo' button, and a 'Continue' button at the bottom.

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The screenshot shows the 'Administration' module of the neuronUP system. The 'Professionals' section is selected in the left sidebar (marked with a red circle '1'). The main area displays a list of professionals and a detailed profile for 'Prof 02'. The 'Patients' modal window is open, showing a search bar and a list of patients working with Alberto (marked with a red circle '2'). The modal also includes a 'Save' button at the bottom.

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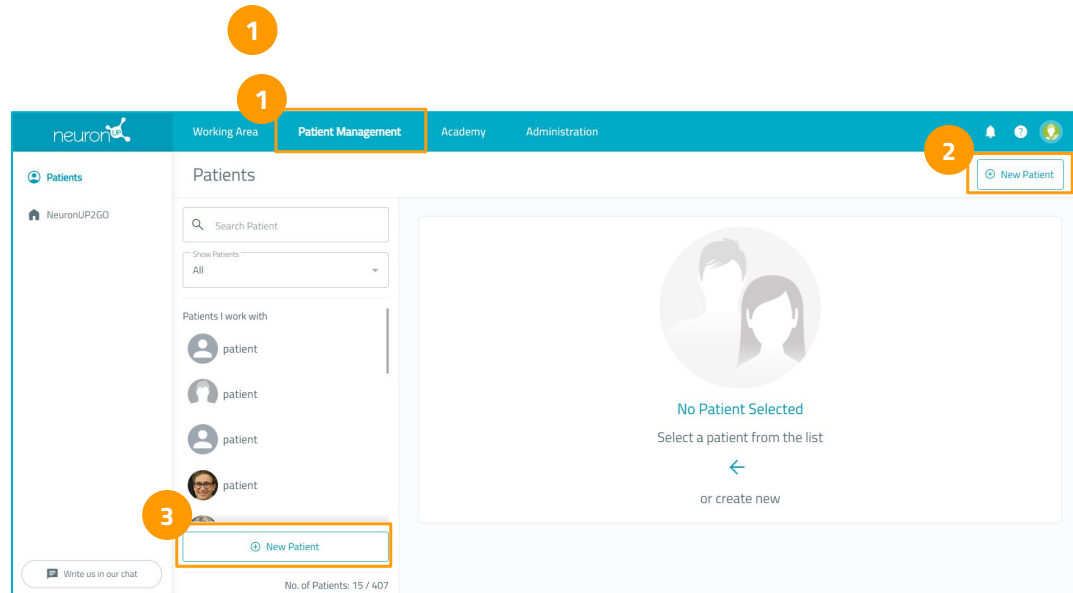
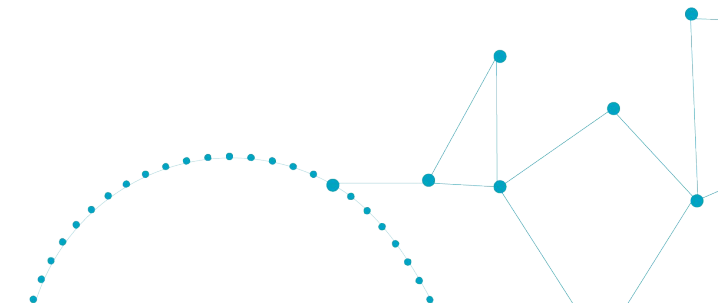
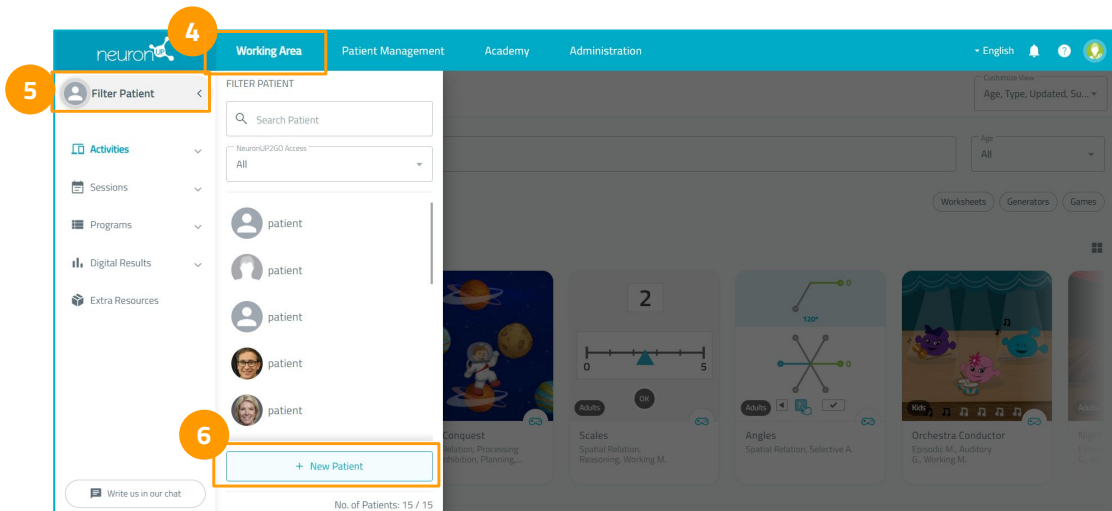
Name*

Surname*

Upload a photo or select a default image

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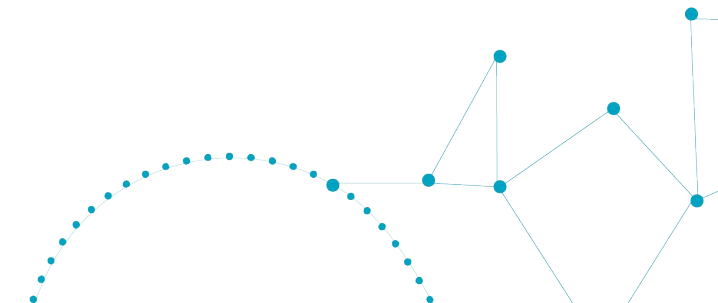
Write Here...

Suggested Diagnoses

- Major neurocognitive disorder due to Alzheimer's disease (DSM5-TR)
- Down syndrome (ICD 11)
- Epilepsy (ICD 11)
- Mild cognitive impairment (Global Deterioration Scale (GDS))

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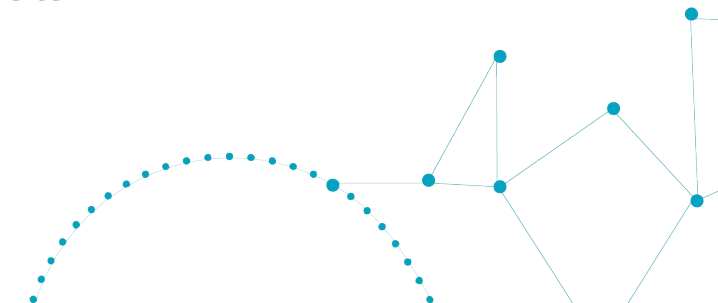
New Patient

1 Basic information — 2 Diagnosis Optional — 3 Relevant information Optional

Gender	Country of residence Spain
Birthdate	Native language Spanish
Education	Dominant hand

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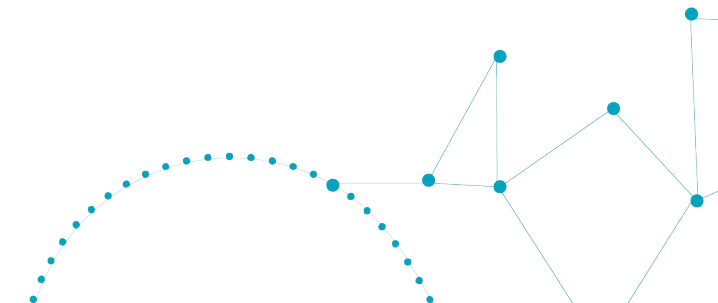
1

Assessment tests

The screenshot displays a grid of assessment tests. A red box labeled '1' highlights the 'Assessment tests' header. A red box labeled '2' highlights the '1 min 50 secs' and '2 subtests' information for the 'Motor Speed Test (Finger Tapping)'. A red box labeled '3' highlights the 'Open test' button for the same test.

Motor Speed Test (Finger Tapping)
Processing Speed
This test is based on the Finger Tapping task described by Strauss, Sherman, and Spreen (2006). A total of 10 trials are planned, 5 with each hand, and the dominant hand is recorded in the initial protocol. It is used as a measure of...

Perceptual-Motor Speed Test (Simple Reaction Time)
Processing Speed, Sustained A., Selective A.
This task is inspired by classic Simple Reaction Time tasks (e.g., Reicker, Tombaugh, Walker & Freedman, 2007), and consists of responding to a stimulus as quickly as possible. It is used to measure basic perceptual...

Response Selection Speed Test
Processing Speed, Decision Making, Sustained A.
This task is inspired by the Choice Reaction Time task by Chiaravallotti, Christodoulou, Demaree & DeLuca (2003). It involves selecting the correct response for each stimulus: respond with the left hand if a square appears...

Visual Search Test with Interference
Inhibition, Selective A., Processing Speed
This task is inspired by one of Neisser's paradigms (1964). It consists of responding with the left hand when the stimulus (a string of six letters) contains the letter 'Z'. Otherwise, respond with the right hand.

Spatial Working Memory Test 1-back
Working M., Processing Speed, Spatial Relation
The 1-back task is a working memory test in which the participant must indicate whether the current location matches the one presented immediately before. It assesses the ability to maintain recently perceived...

Spatial Working Memory Test 2-back
Working M., Processing Speed, Spatial Relation
The 2-back task is a working memory test in which the participant must indicate whether the current spatial location matches the one presented two positions earlier. It assesses the ability to retain visuospatial...

Motor Speed Test (Finger Tapping)
Processing Speed
This test is based on the Finger Tapping task described by Strauss, Sherman, and Spreen (2006). A total of 10 trials are planned, 5 with each hand, and the dominant hand is recorded in the initial protocol. It is used as a measure of...

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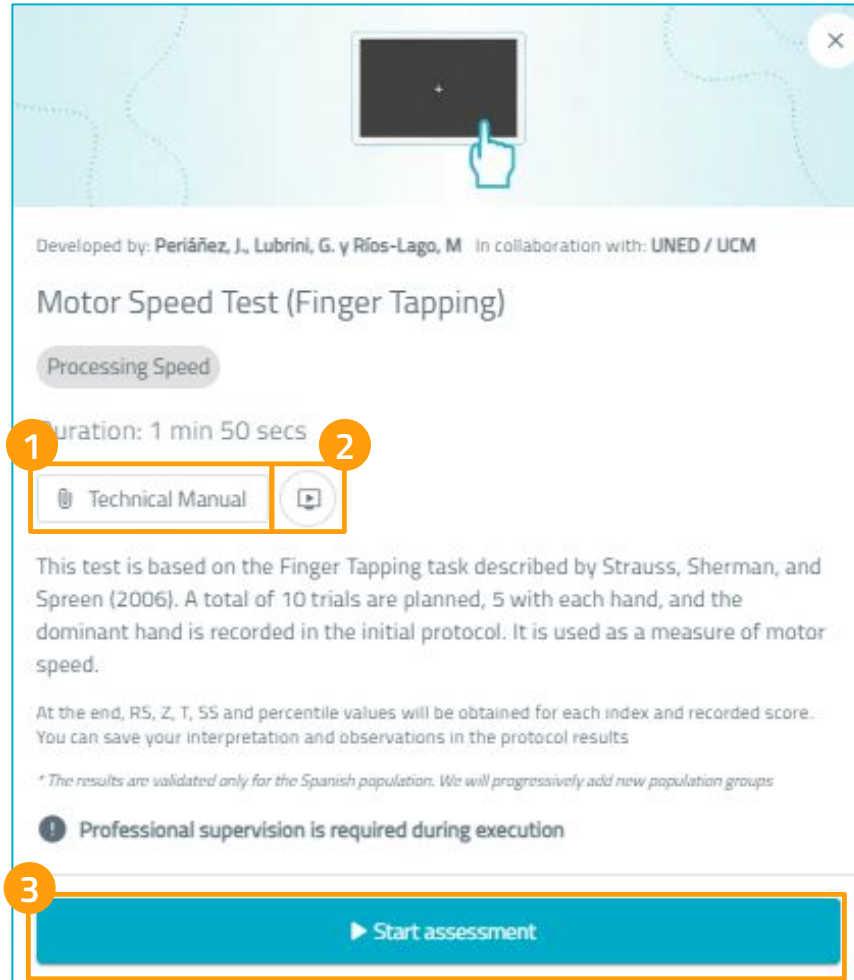
Duration: 1 min 50 secs

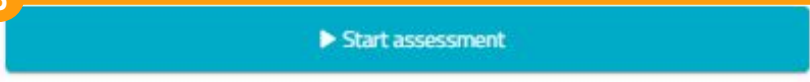
1  Technical Manual  2

This test is based on the Finger Tapping task described by Strauss, Sherman, and Spreen (2006). A total of 10 trials are planned, 5 with each hand, and the dominant hand is recorded in the initial protocol. It is used as a measure of motor speed.

At the end, RS, Z, T, SS and percentile values will be obtained for each index and recorded score. You can save your interpretation and observations in the protocol results

** The results are validated only for the Spanish population. We will progressively add new population groups*

 Professional supervision is required during execution

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< Pre-assessment form

 Assessment of 'André Ferreira' with 'Go-No Go Test (SART)'
Please complete the following questions. This will help us provide reliable indexes

1 How will you interact with the application during the test?
Remember that using a mobile phone, projectors, or screens larger than 27 inches is not advisable

With keyboard and mouse With touch screen

2 User's mood
Ask the user to rate their mood on a scale from 1 (very bad) to 7 (very good)

1 2 3 4 5 6 7

How the user slept last night
Ask the user to rate their night rest on a scale from 1 (very bad) to 7 (very good)

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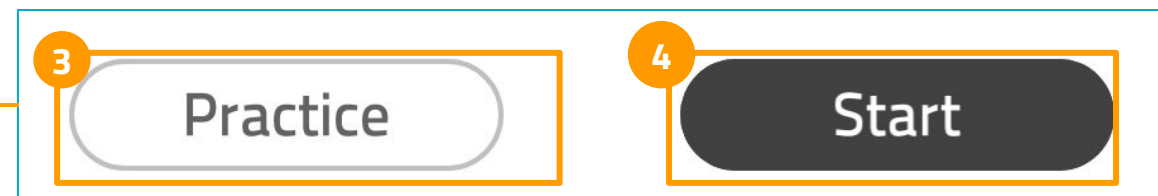
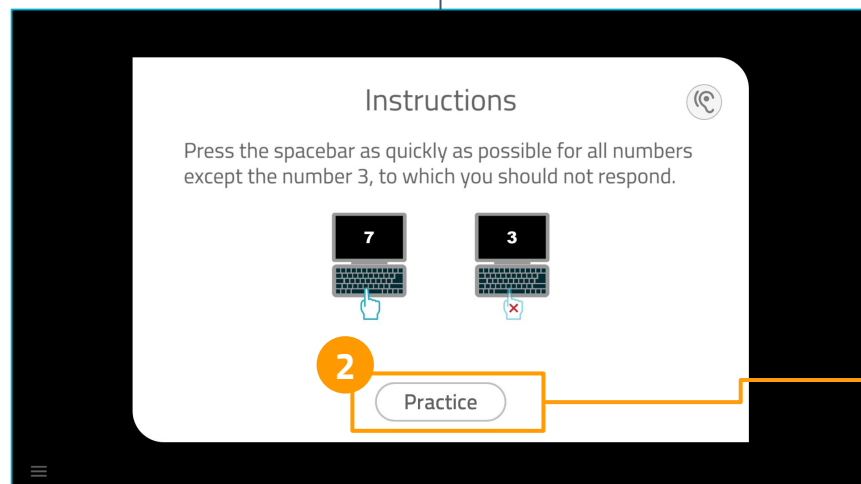
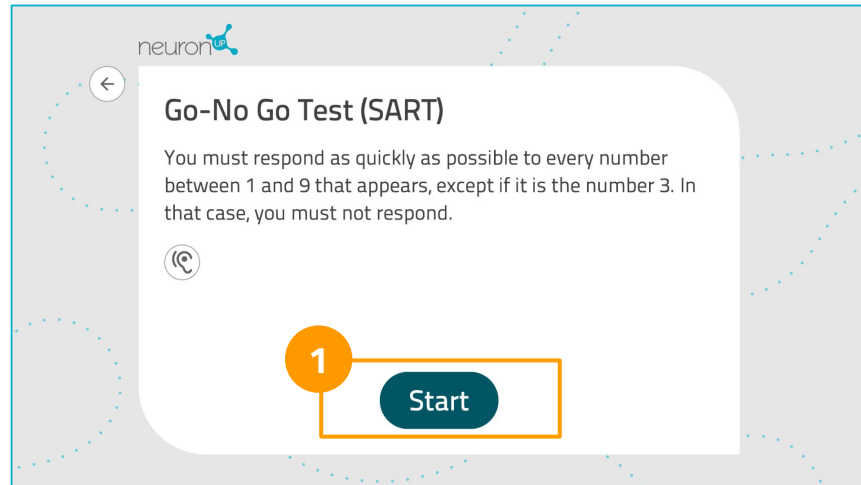
Continue

Post-assessment form

Assessment of 'André Ferreira' with 'Go-No Go Test (SART)'
Please complete the following questions. This will help us provide reliable indexes

How well do you think you did on the test?
Ask the user to rate their subjective sense of performance on a scale from 1 (very poor performance) to 7 (excellent performance)

1 2 3 4 5 6 7

How tired are you?
Ask the user to rate their feeling of fatigue on a scale from 1 (extremely tired) to 7 (not tired at all)

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The screenshot displays the 'Assessment' section of the neuronUP platform. The top navigation bar includes 'Working Area', 'Client Management', 'Academy', and 'Administration'. The user 'André Ferreira' is logged in. The main content area is divided into 'Assessment batteries' and 'Assessment tests'.

Assessment batteries:

- SPAIN (Speed of Processing Assessment Instrument):** Battery composed of 10 reaction time tasks aimed at assessing information processing speed and cognitive efficiency. Includes measures such as mean response time, speed-accuracy. (30 mins, 20 secs, 10 tests)
- Perceptual-Motor Speed Battery (SPAIN-PMS):** Battery consisting of 3 tasks designed to evaluate perceptual and motor speed. Measures the speed of perceiving stimuli and issuing motor responses. Analyzes reaction times, motor fatigue, and... (6 mins, 50 secs, 3 tests)
- Working Memory Battery (SPAIN-WM):** Battery consisting of 4 tasks that evaluate verbal and spatial working memory. Assesses the ability to retain, manipulate, and update information in real time. Includes repetition metrics... (10 min, 4 tests)

Assessment tests:

- Motor Speed Test (Finger Tapping):** Processing Speed. This test is based on the Finger Tapping task described by Strauss, Sherman, and Spreen (2006). A total of 10 trials are planned, 5 with each hand, and the dominant hand is recorded in the initial protocol. It is... (1 min, 50 secs, 2 subtests)
- Perceptual-Motor Speed Test (Simple Reaction Time):** Processing Speed, Sustained Attn., Selective Attn. This task is inspired by classic Simple Reaction Time tasks (e.g., Reicker, Tombaugh, Walker & Freedman, 2007), and consists of responding to a stimulus as quickly as possible. It is used to measure basic perceptual... (3 min)
- Response Selection Speed Test:** Processing Speed, Decision Making, Sustained Attn. This task is inspired by the Choice Reaction Time task by Chiaravalloti, Christodoulou, Demaree & DeLuca (2003). It involves selecting the correct response for each stimulus: respond with the left hand if a square... (2 min)
- Go-No Go Test (SART):** Inhibition, Processing Speed, Selective Attn. This task is based on the SART paradigm (Robertson et al., 1997) and is used to assess response inhibition strategy. It is a Go-No-Go task. (1 min, 30 secs)
- Visual Search Test with Interference:** Inhibition, Selective Attn., Processing Speed. This task is inspired by one of Neisser's paradigms (1964). It consists of responding with the left hand when the stimulus (a string of six letters) contains the letter 'Z'. Otherwise, respond with the right... (4 min)

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The screenshot illustrates the steps to start an assessment battery in the neuronUP software. It shows the 'Filter Client' button (1) in the sidebar, the 'Assessment' section for a selected user (2), and the 'Open battery' button (3) for the 'SPAIN (Speed of Processing Assessment Instrument)' battery. Below this, there are several individual test cards, each with an 'Open test' button.

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The screenshot shows the 'Assessment batteries' section in the neuronUP interface. Three battery cards are visible: SPAIN (Speed of Processing Assessment Instrument), Perceptual-Motor Speed Battery (SPAIN-PMS), and Working Memory Battery (SPAIN-WM). The SPAIN card has a red '1' and an 'Open battery' button highlighted. A red box labeled '2' points to the 'Technical Manual' button in the expanded view of the SPAIN battery. The expanded view displays the following information:

- Developed by: Periañez, J., Lubrini, G. y Ríos-Lago, M. In collaboration with: UNED, UCM
- SPAIN (Speed of Processing Assessment Instrument)
- Working M. Processing Speed Selective A.
- Duration: 30 mins 20 secs
- Technical Manual
- Battery composed of 10 reaction time tasks aimed at assessing information processing speed and cognitive efficiency. Includes measures such as mean response time, speed-accuracy trade-off, and restart cost.
- At the end, R5, Z, T, S5 and percentile values will be obtained for each index and recorded score. You can save your interpretation and observations in the protocol results
- * The results are validated only for the Spanish population. We will progressively add new population groups
- Professional supervision is required during execution
- Continue assessment

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1 Continue

Post-assessment form

 **Assessment of 'André Ferreira' with 'Go-No Go Test (SART)'**
Please complete the following questions. This will help us provide reliable indexes

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Assessment protocol content

Choose the test you want to continue with

Motor Speed Test (Finger Tapping)
1 min 50 secs 2 subtests

Perceptual-Motor Speed Test (Simple Reaction Time)
3 min

Go-No Go Test (SART) ✓ Completed
1 min 30 secs

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Verbal Working Memory Test 2-back
2 mins 30 secs

Spatial Working Memory Test 1-back
2 mins 30 secs

Spatial Working Memory Test 2-back
2 mins 30 secs

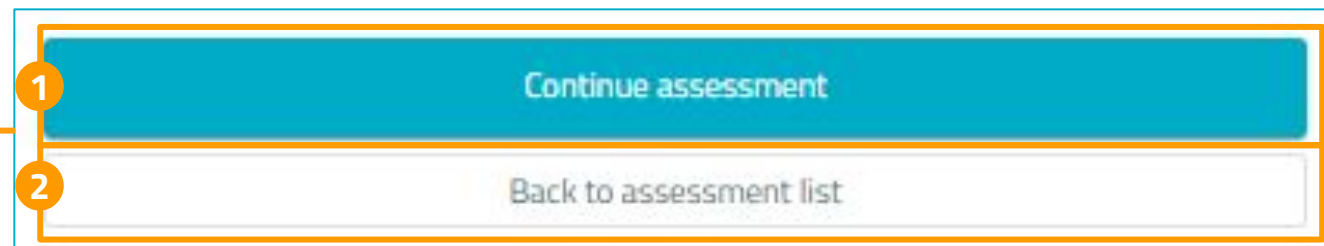
Cognitive Flexibility Test (task switching)
8 min

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Active assessment protocol
Since January 23, 2026
Started by: Carmen -

Tests to complete: 9

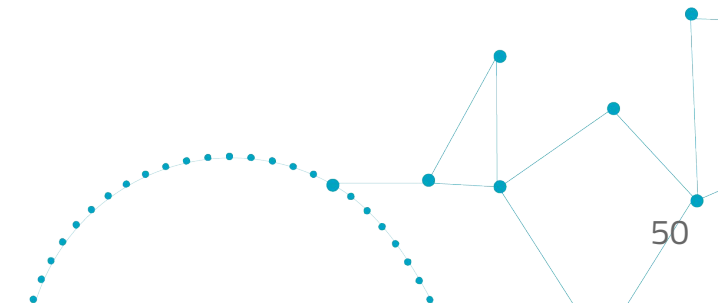
- 1 View results
- 2 New Report
- 3 Finish assessment
- 4 Continue assessment

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Evaluationsbatterien

Active assessment protocol
 Since January 23, 2026
 Started by: Carmen -

Tests to complete: 9

1 View results New Report Finish assessment Continue assessment

Assessment results Active protocol

Assessment protocol result for 'Manuel Garcia'
 Since Jan 29, 2026, 3:23:32 PM
 Started by: Carmen -
 2 tests finished

Go-No Go Test (SART)
 Date: Feb 2, 2026, 10:26:59 AM

Score	RS	Z	T	SS	PR
Correct answers	0	-3.5	15	1	5
Omissions	45	-3.5	15	1	5
False alarms	0	3	90	19	99
RT fatigue in correct responses	0	0	50	10	49

2 Score interpretation

3 Pre-assessment form
 How will you interact with the application during the test? With keyboard and mouse
 User's mood 4 / 7
 How the user slept last night 4 / 7
 Post-assessment form

4 Assessment observations

5 Generate interpretation with +:AI PRO

6 The results of this test and their AI-based interpretation are exclusively exploratory in nature and are intended to support research and professional reflection. They do not constitute a diagnosis and should not be used as the sole basis for clinical intervention

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The screenshot shows the 'Active assessment protocol' interface. At the top, there's a header with 'Active assessment protocol' and 'January 29, 2026'. Below that, a 'Tests to complete: 2' indicator is shown. The main area displays 'Assessment tests' with a grid of test cards. Each card has a 'Test in active protocol' label (3) and an 'Open test' button (1). A 'Continue assessment' button (4) is located at the top right. A dialog box titled 'Assessment protocol content' is open, showing a list of tests: 'Torre de Hanói' (Completed), 'Go-No Go Test (SART)' (Completed), 'Stroop Test' (selected), and 'Symbol Digit Modalities Test (SDMT)'. A 'Continue' button (5) is at the bottom of the dialog.

6. Ergebnisse

Ergebnisse

The screenshot shows the 'Client Management' section of the NeuronUP software. The interface includes a top navigation bar with 'Working Area', 'Client Management', 'Academy', and 'Administration'. On the left, there is a 'Clients' sidebar with a search bar and a list of clients: André Ferreira, Daniela Cruz, Julen López, Manuel García, and Manuel Sánchez. A 'New Client' button is at the bottom of the list. The main area displays 'Assessment results' for 'NeuronUP2GO'. It lists four assessment protocols with their respective dates and test counts. Each protocol has a 'View results' button and a trash icon. At the bottom, there is a 'Reports' section with a 'New Report' button. Numbered callouts (1-6) highlight the 'Client Management' tab, the client list, the 'Assessment results' section, the 'View results' buttons, the trash icons, and the 'New Report' button.

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7. Informes

Berichte

The screenshot shows the 'Client Management' interface. At the top, the 'Client Management' tab is selected (1). On the left, there is a search bar and a list of clients. The client 'André Ferreira' is highlighted (2). The main area displays 'Assessment results' for 'NeuronUP2GO'. It lists four assessment protocols with their respective test counts and dates. A 'New Report' button is highlighted in the bottom right (3).

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Berichte

The screenshot shows a 'New Report' form with a progress bar at the top indicating four steps: 1 Anamnesis, 2 Content, 3 Functions, and 4 Conclusions. The form is divided into two main sections: 'Basic information' and 'Additional information'. The 'Basic information' section includes a 'Report Date' field (1) with the value '01/23/2026', a user profile icon for 'André Ferreira', a 'Report Name*' field (2) with a subtext 'Assign a descriptive name to the PDF file to easily identify its content in the future.', and an 'Anamnesis*' field (3) with a subtext 'Add a brief introduction with relevant information or the purpose of the report.'. At the bottom of the form is a blue 'Continue' button (5).

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This section provides a detailed view of the 'Additional information' part of the form. It contains two text input fields: 'Observations Title' and 'Used tools', both grouped under a single header. Below these is a larger text area for 'Observations'. A callout box with the number 4 points to this section.

Berichte

New Report

1 Anamnesis 2 Content 3 Functions 4 Conclusions

NeuronUP Report Information:

Required Information

Relevant information

Diagnosis

Assessment results:

Select the results of the evaluation protocols you want to include in the report

Since 1/23/26 Active protocol

Assessment protocol 1
1 test finished View results

From 1/23/26 to 1/23/26

Assessment protocol 2
1 test finished View results

From 1/22/26 to 1/23/26

Assessment protocol 3
View results

Rehabilitation results:

Interval: Last 3 Months Group By: Day

Number of activities worked, days worked, hours worked

Show impact of cognitive functions on the score

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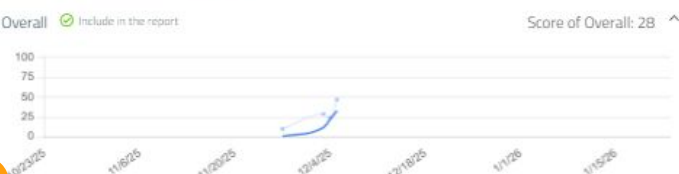
New Report

1 Anamnesis 2 Content 3 Functions 4 Conclusions

Cognitive functions worked:
Select the functions you want to analyze so they appear in the report

Generate analysis with +: AI 

Overall Include in the report Score of Overall: 28 ^

1 Analysis

2 Include in the report

Processing Speed Include in the report Score of Processing Speed: 13 v

Orientation Include in the report Score of Orientation: 50 v

Attention Include in the report Score of Attention: 12 ...

Continue

1 Anamnesis 2 Content 3 Functions 4 Conclusions

Conclusions:
Use this field to note any elements you believe are of interest; observations, comments, analysis on user evolution or interpretation and recommendations.

4 Conclusion Title
User follow-up

5 Report Conclusions

6 Generate conclusions with +: AI 

7  This content has been generated by an automated system. It does not replace professional judgment nor constitute a diagnosis. Review and validation by the professional responsible for the case are required.

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